


Caretech Transmitter

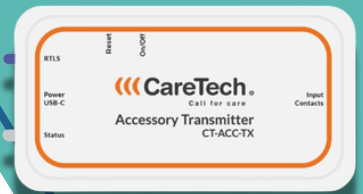

1

Turn on the transmitter, checking what colour light emits from the status LED. If the LED has a **green** light, proceed to step 3. If the LED has a **red** light, proceed to step 2.

2

If the status LED has a **red** light turn the transmitters off. While holding the reset button turn the transmitter back on. When you let go of the reset button the light will change to **red**. If the light is still **green**, you will need to reattempt this step until it is successful.

CONNECTING TO OTHER PRODUCTS

3

Open the pouch on the sensor product and plug the cord into the jack on the right side of the transmitter.

4

You can test the bed and chair mats by applying pressure on them for approximately 7 seconds, the status LED on the transmitter will **flash green** once, indicating it is armed. Now release pressure from the mat and you will see the transmitter **flash red** for 5 seconds and then the alarm will transmit.

POWERED BY: **CareTech**
Call for care