

## PIR



1

Power on the PIR with the ON/OFF switch.



2

Plug in the 3.55mm jack cord provided into the socket on the side.



3

Turn the transmitter on and plug the other side of the cord into the transmitter.

4

Mount the PIR, after 30 seconds, test it by moving past it and seeing a **red** flash/alarm trigger.

5

If necessary, adjust the sensitivity using the potentiometer by turning the dial left to decrease sensitivity or turn it right to increase sensitivity.

POWERED BY:  **CareTech**  
Call for care