

Pairing Receiver to Transmitter



1

Power on the transmitter with the switch on the top of the device. Check the colour that flashes when powering on. If the LED flashes **red**, proceed to step 3.



2

If the LED flashes **green**, turn the transmitter off, then while holding the reset button, turn the transmitter back on. The LED will change to **red** once you let go of the reset button.



3

Power on the receiver with the switch on the side, indicated by the on/off sticker.

4

Hold the pair button on the receiver for 4 seconds or until the LED on the front **flashes green**. The LED will blink for 3 seconds. While the LED is still blinking, press the reset button on the transmitter.



5

The LED on the receiver will turn **solid green** and then stop, and the LED on the transmitter will rapidly **blink red** several times and then stop. **Your devices are now paired!**

LED STATUS LIGHT - RED - This mode is for pairing to a receiver.
GREEN - This mode is for pairing to a CareTech system.

CHARGING - The recommended charging time is 2-4 hours.
 Devices can last 1-3 months once charged, dependant on usage.

POWERED BY: **CareTech**
 Call for care