

RTM Floor Mat - Wireless



1

Ensure your transmitter and receiver are paired. Check out our 'Pairing an accessory transmitter and accessory receiver' for these instructions.



2

Ensure your transmitter is in red mode indicated by a **red** LED when you turn the transmitter on. If the field is **green**, turn the transmitter off. While holding the reset button, turn the transmitter back on. When you let go of the reset button, the LED will change to **red**.



3

Grab your floor mat and the 4-pin adapter. At the back of the floor mat there is a pouch, inside that is a 4-pin cable. Plug the adapter into the mat.



4

Plug the receiver into the callpoint in the room you want to use the floor mat in.

5

Test the floor mat by applying pressure to it for 3-5 seconds and checking to see if the alarm was sent to the nurse call system. **Your floor mat is now set up wireless.**

POWERED BY: **CareTech**
Call for care